
Breast Pump


- | | |
|---|--|
| EN Electric Breast Pump | HU Elektromos mellszívó |
| PL Laktator elektryczny | RO Pompă de sân electrică |
| DE Elektrische Milchpumpe | SK Elektrická odsávačka materského mlieka |
| FR Tire-lait électrique | LT Elektrinis krūties siurblys |
| IT Tiralatte elettrico | UA Електричний молоковідсмоктувач |
| ES Extractor de leche materna eléctrico | BG Електрическа помпа за кърма |
| CZ Elektrická odsávačka mateřského mléka | GR Ηλεκτρικό θήλαστρο |

I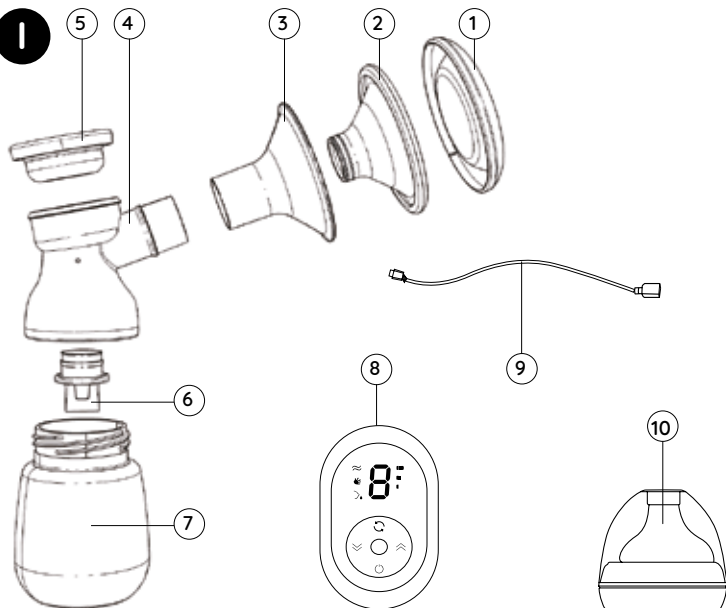

II

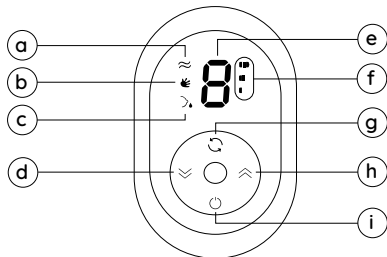
III

1

2

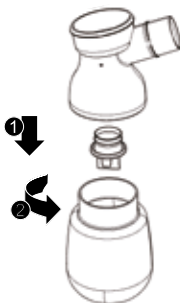
3


4

5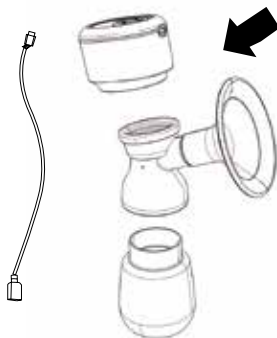

IV

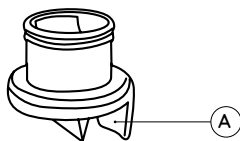Thank you for purchasing this Nukido product, which has been designed with your child's safety in mind. To ensure safety and trouble-free use, please read the following warnings and instructions for use, follow them strictly and keep them for future reference.

Introduction

- Breast milk is the healthiest and most valuable food for newborns and babies, needed for proper growth and development. It contains many antibodies that protect your baby from infections and allergies. Breastfeed for the first 6 months of your baby's life. After the 6th month combine breast milk with solids.
- Caution: do not use a breast pump if your baby cannot empty the breast due to blockage of the milk ducts. The breast pump is designed to express excess milk and stimulate the sucking process. If you have feeding problems or pain, consult your doctor before using a breast pump.
- The breast pump is a personal piece of equipment and is not exchangeable or returnable. Remember that use by more than one person can pose serious health risks.
- This automatic breast pump has a unique design for more comfortable pumping. The soft massaging cushion provides a pleasant sensation, reminiscent of a baby's sucking, allowing the milk to flow out quietly, gently and quickly. The breast pump is compact and easy to assemble. All parts are dishwasher safe. The breast pump does not contain bisphenol A (BPA Free).
- The device can help to extend breastfeeding time. You can express and store your milk, so your baby can still enjoy your milk even if you cannot breastfeed directly.
- The breast pump is compact and very quiet, so you can carry it with you and pump at any convenient time.

Warnings!

- Do not attempt to repair or modify the product yourself, this poses a serious risk to health and life and will void your warranty.
- Use the breast pump as intended and according to the instructions. Improper use may endanger the user and damage the product.

- The first use of the breast pump should be under the supervision of a healthcare professional and should follow your doctor's instructions.
- Adjust the suction pressure correctly. Too little suction can reduce the amount of milk being pumped and too much suction can cause pain and reduce the amount of milk produced.
- Inadequate suction strength and incorrect use of the breast pump can lead to breast soreness or injury.
- Keep the product out of the reach of babies and small children.
- To protect the product from damage, avoid dropping or bumping it.
- This product has a built-in lithium battery. When the device shows a low battery, recharge the battery as soon as possible using a certified charger.
- Do not attempt to replace the battery yourself. Some battery components are flammable and harmful, which may cause injury, fire, rupture or explosion.
- Do not disassemble, open, shred or puncture the battery.
- Do not overcharge batteries.
- Have lithium batteries replaced only by authorised service technicians.
- Remember to dispose of batteries according to local regulations.
- Do not touch the breast pump with bare hands immediately after sterilisation in boiling water. Risk of scalding.
- A specific odour is a natural phenomenon. To reduce the intensity of the odour, wash the breast pump thoroughly according to the instructions.
- For your baby's safety and health, always use this product under adult supervision. Never leave your baby unattended while feeding.
- Never use bottle teats as soothers.
- Prolonged continuous sucking can affect dental disease.
- Always check the temperature of the food before feeding.
- Keep unused parts out of reach of children.
- Take care when heating food. Always stir heated products to ensure even heat distribution and check the temperature before serving to avoid scalding.
- Check the teat before each use by pulling it in all directions. If you notice the first signs of damage, throw the soother away.
- Do not expose the soother to direct sunlight or high temperatures. Do not leave the soother in the disinfectant for longer than recommended so as not to weaken the product.
- Before first use, place the parts in contact with the food in boiling water for 5 minutes.
- Before each use, thoroughly wash all breast pump parts according to the instructions.

- Do not wash the pump motor in the dishwasher.
- Do not immerse the motor and power pack of the device in water or wash with a wet sponge or wet cloth.
- Do not use the product in a humid environment.
- Use the breast pump indoors only. Do not use the breast pump outdoors. The device is suitable for domestic use only.
- If the device has been transported in cold temperatures, acclimatise it before use to equalize the temperature of the device with the environment.
- Do not leave the breast pump near heat sources such as radiators, sunny windows.
- Do not immerse the cable, power supply or breast pump motor in water or any other liquid.
- Protect the motor from getting wet. Never operate the appliance with wet hands.
- Connect the appliance to an electrical outlet with the voltage as shown on the appliance's rating plate.
- Do not leave the appliance unattended when it is plugged in.
- Never pull the power plug out of the socket by pulling on the cable.
- Do not place power cords under furniture or other obstructions.
- Protect the mains cable from damage, keep it away from heat sources and sharp edges.
- Disconnect the product from the power supply before cleaning.
- Do not use the product if the mains cable is damaged. If the cable is damaged, have it replaced by an authorised service centre.
- Do not operate the appliance if the motor housing is visibly damaged.
- Only trained personnel should repair the appliance. Improperly performed repairs can cause serious hazards for the user.
- The equipment may be used by persons with reduced physical and mental capabilities and persons with inexperience and lack of knowledge of the equipment, if supervision or instruction is provided on how to use the equipment safely and if the risks involved are understood.

Components of the equipment (Figure I)

1. Dust cover
2. Massaging pad
3. Attachment
4. Body
5. Silicone membrane

6. Valve
7. Feeding bottle
8. Pump motor
9. Cable and adapter
10. Nipple

Control panel (Fig. II)

- a. Suction mode indicator
- b. Massage mode indicator
- c. Indicator of stimulation mode
- d. Button for reducing power
- e. Power indicator
- f. Battery status indicator
- g. Change mode button
- h. Increase power button
- i. Power button

Pre-installation instructions

- Before first use, clean and disinfect all components according to the section "Cleaning and disinfection".
- Disinfect all components before each use.
- Wash your hands thoroughly before touching cleaned components. Remember that scalded components may still be hot.
- Disinfect all breast pump components and wash your hands thoroughly before assembly.

Note: assembly may be easier when the breast pump is wet.

Assembly (Fig. III)

1. Insert the valve into the bottom of the body by grasping the valve by the side handle A (see fig IV).
2. Screw the body onto the feeding bottle as far as it will go.
3. Place the silicone diaphragm in the upper part of the body. Tighten the diaphragm to make sure it is firmly fitted.
4. Place the pump motor in the top part of the body. Tighten it to make sure it is stably mounted.

5. Insert the massage pad into the breast shield funnel in the body, push in and make sure the pad is snugly fitted. Press the petals to remove any residual air. Finally, connect the power supply to the motor.

Tips for use

Wait with expressing milk until lactation and feeding time have normalised, which is about 2 to 4 weeks after the baby is born.

Here are the exceptional situations:

- When you need to feed your baby in hospital.
- If your breasts are sore or engorged, you can express a small amount of milk before a feeding or between two feeds to ease the pain and make it easier for your baby to suckle.
- If you have a problem with your nipple, you can use a breast pump once the problem has subsided.
- If you are temporarily separated from your baby but wish to continue breastfeeding after a break, you can pump regularly to stimulate milk secretion.

Time and conditions for expressing

- Choose the best time of day to express, such as in the morning when milk is plentiful, before or after your baby's first feed or when your baby does not want to drink breast milk. You can express milk during your break.
- Choose a time when you don't have to rush and nothing will distract you. Pictures of the baby can support lactation. A warm environment also helps to express milk: try expressing after a bath or place a warm towel on your breast for a few minutes before expressing.
- If you are experiencing lactation pain or have questions about this, consult your lactation consultant.

Button functions and meaning of indicators

Press the power button to switch on the machine. The breast pump will turn on in the default mode. Press the change mode button to switch between modes.

- **Massage mode** (9 levels in total)
- **Stimulation mode** (9 levels in total)
- **Extraction mode** (9 levels in total)

Each time you click, the current level is displayed on the screen.

The device switches off automatically after 30 minutes.

The battery indicator will flash while charging, once fully charged the icon will be on solid light.

Full instructions for use

1. Wash your hands thoroughly and make sure your breasts are clean. Relax and sit in a comfortable place (you can use a cushion for back support).
2. Gently hold the breast, place the cover over the body and make sure the breast is centred and the massaging breast pad forms a seal.
3. Press the power button, the screen will light up on start-up, the pump will automatically start in default mode. Press the increase power button or the decrease power button to select the massage strength. When the milk starts to flow, press the change mode button to switch to pumping mode, select the most comfortable suction power and pumping frequency.

Note: you do not have to use the highest power, use the one you find comfortable.

Note: Do not overfill the bottle above the valve.

4. When you have finished pumping, switch off the breast pump and carefully unscrew the bottle from the breast pump body to prepare it for feeding/storage. Follow the instructions in the "Cleaning and disinfection" section to clean the used parts of the breast pump.
5. Disconnect the motor.

Cleaning and disinfection

- Clean and disinfect the breast pump before first use.
- Disassemble the breast pump and clean all its parts after each use.
- Remember that washing the motor is strictly prohibited! You can wipe this part with a clean soft cloth.
- It is important to clean the valve. If the valve is damaged, the breast pump will not work properly. To remove the valve, carefully pull the valve handle. Gently wipe the valve with clean water and a few drops of cleaning fluid, do not insert any objects into the valve.
- Clean all parts with hot water, below 100°C, with a small amount of detergent, then rinse thoroughly.

- Remember that silicone parts are not suitable for prolonged disinfection at high temperatures.
- Use a sterilising dish to disinfect all parts or boil in boiling water for 5 minutes.

Storing and warming breastmilk

Storage of breast milk

When you have finished expressing, seal the storage container tightly and immediately start refrigerating or freezing the breastmilk. Breast milk can be stored in the refrigerator for 48 hours and in the freezer for 3 months.

Tips for freezing breastmilk

- If you want to freeze expressed breastmilk, remember not to fill the storage container completely.
- Fill the container 3/4 full, bearing in mind that the ice will increase in volume relative to the liquid.
- Do not place the container on the side door of the fridge.

Tips for defrosting breastmilk

- To preserve the nutrients in the milk, place the frozen milk in the fridge overnight to thaw.
- Gently shake the container so that the fat that has separated during freezing is evenly mixed into the milk.

Heating the milk

Once thawed, warm the breast milk using a dedicated heater or warm water (below 37° C). The warmed milk can be fed to the baby.

Caution!

- Do not use a microwave oven or boiling water to heat, which will lead to a loss of valuable nutrients.
- Do not put frozen milk in the microwave or in boiling water to avoid nutrient loss and to prevent the risk of scalding your baby.
- Do not refreeze milk that has already been defrosted once.
- Never mix fresh milk with thawed milk.

Solving common problems

No suction	• Check that the pump has been correctly assembled. Check that the valve is not damaged (when cleaning, do not rub it to avoid damage). • Check that the silicone diaphragm and pads are securely attached to the breast pump body to ensure a perfect seal.
Unable to express breastmilk	• Make sure the breast pump has adequate suction and that it is fully assembled. • Relax and try again. After a few attempts you may be able to express milk successfully. • Go back to the instructions for pumping.
Breast pain during pumping	• The reason may be that the suction force is too high. You do not need to use maximum suction force to express milk. • First use the massage mode at a high level to stimulate your breasts and then switch to the milk extraction mode. • You can also consult a lactation consultant.

INFORMATION FOR USERS ON HOW TO DISPOSE OF ELECTRICAL AND ELECTRONIC EQUIPMENT.

If products are labelled with a crossed-out wheeled bin, this means that they are subject to selective collection in accordance with the provisions of Directive 2012/19/EU of the European Parliament and of the Council. Used products should not be disposed of with other household waste and should not be treated as municipal waste. Putting electronic waste in the container can be hazardous for the environment. Used products should be taken to public collection points. By sorting waste electrical and electronic equipment and submitting it for treatment, recovery, recycling and disposal, you protect the environment from pollution and contamination, contribute to the reduction of the level of exploitation of natural resources and lower the costs of production of new equipment. Proper disposal and scrapping help eliminate the negative effects of scrapped products on the environment and human health. For more detailed information about the recyclability of this product, please contact your city office, municipal cleaning services or the store where you purchased the product.

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© All rights reserved by Global Income.